Bleach'n Smile

Chair-side Bleaching Gel with
neutral pH-value

**Bleach'n
Smile**

SCHÜTZ  DENTAL
GROUP

Appreciating beautiful teeth?

With the extended bleaching line, Schütz Dental offers the perfect product for everyone. The neutral pH-value and the Bleach'n Smile After Bleaching Care Cream support the remineralization and protect the enamel.

Advantages of Bleach'n Smile Automix:

- easy to use
- quick penetration by 35 % hydrogen peroxide content (after mixing)
- gentle on natural teeth (pH-neutral value 7.3)
- safe storage (no segregation, no bubbles because of a safety clip)
- safe (CE-certified)

Self-activating

The effective formula makes Bleach'n Smile Gel ideal for chairside use. Laser- or light-activation is not necessary.

Plasma bleaching

Bleach'n Smile Automix can also be light-activated (plasma lamp), which helps to reduce the treatment time.

Note:

Furthermore, Bleach'n Smile can be laser-activated, which helps to speed up the bleaching process and to influence the bleaching result significantly.

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6 Steps towards brighter teeth

Determine the shade of the original tooth before starting the treatment.

Apply vaseline to the lips. Apply a lip and cheek retractor. Apply and polymerize the Bleach'n Smile Dental Dam for gingiva protection.

Apply the self-mixing Bleach'n Smile Gel onto the labial surfaces of the teeth (a film approx. 1-2 mm thick) and allow for 10 minutes residence time.

The bleaching process can be accelerated by light or heat, if desired.

After 10 minutes, remove the gel and rinse the teeth. The procedure can be repeated up to two times, if necessary.

Apply Bleach'n Smile After Bleaching Care Cream to remineralize the natural teeth.

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The advantages at a glance

Bleach'n Smile Automix

Bleach'n Smile is a bleaching gel developed solely for chairside use. A double cartridge for instant mixing contains bleaching gel and activator. Mixed gel contains 35 % of hydrogen peroxide. Including dental dam (light curing gingiva protector) and cream for the remineralization of the teeth.

Special formula with potassium nitrate and sodium fluoride to avoid sensitivities!

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