Product Innovations

Dental - Labor | Dental Laboratory | Laboratoire Dentaire | Laboratorio Dentale | Laboratorio Dental
Zahnarzt - Praxis | Dental Practice | Cabinet Dentaire | Pratica Dentale | Práctica Dental

edenta | 

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All rights reserved.

Reproduction, also by extract and reproducing photos are only permitted with written authorization of EDENTA AG, Switzerland. All offers, orders and deliveries are subject to Edenta's „General sales and Delivery Terms“. We reserve the right to modify our range of products and their design as well as to deviate from the illustrations and data shown.

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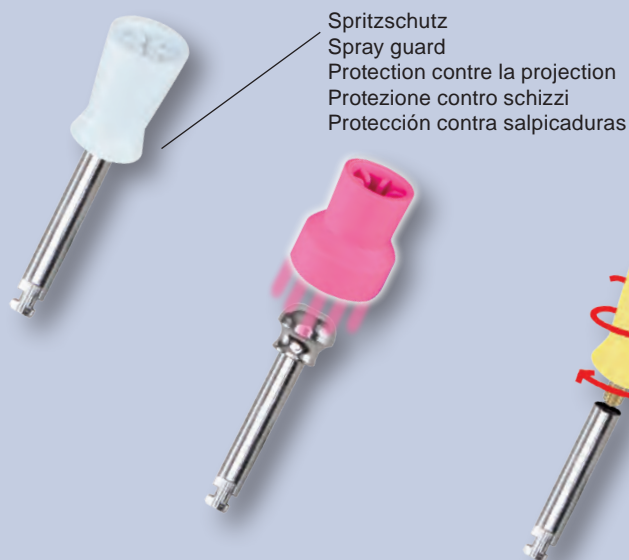
Inhaltsverzeichnis | Index | Sommaire | Indice | Índice

	Page
Prophylaxis (Polishers)	4 / 5
Prophylaxis (Brushlets)	6 / 7
TopBrush (SiC-Brushlets)	8
PowerCut TC Bur (AmalgamRemover & CrownCutter)	9
K-Diamonds (CrownCutter & Trepanation on All Ceramic)	10 / 11
Selective Dentin Remover (C1SDR.204...)	12
Bambino-Diamonds (Micropreparation)	13
S-Finisher (Composite Restoration)	14 / 15
EndoAccess Burs (H1SNL...& H152...)	16
Crown Prep-Set (FG-Diamonds)	17
Soft Tissue Trimmer (STT 250...)	18
Bur Blocks (universal for FG-RA-HP Instruments)	19
All-Ceramic Polishing (CeraGloss-FLEX)	20
TC-Cutter (VZ27 new toothing for working NEM)	21
All-Ceramic CAD/CAM Restoration (Diamond Disc 327...)	22
Press-Ceramic (Diamond Disc 378... for pressceramic technique)	23

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Prophylaxis treatment, maximum cleaning and polishing of the tooth surface

Healthy teeth and healthy gingiva – with the correct prophylaxis. The new prophylactic polishers with improved polishing properties smooth the tooth surfaces to prevent renewed bacterial attachment. The polishers are gentle on the tooth structure and greatly reduce the uncomfortable heat build-up in the teeth. This ensures thorough and reliable tooth cleaning. Colour coding indicates the different hardness of the polishers: white = standard polishers for removing dental plaque and polishing, pink = soft polishers for removing dental plaque in the region of the sulcus, to protect the soft tissue, yellow = hard polishers for removing heavy dental plaque and discoloration. For single use, non-autoclavable.

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Polisher with screw thread and spray guard for screw retention in all known prophylaxis contra-angle handpieces or in the reusable screw mandrel 4039RA. Optimal motor speed = 1,500 rpm. Colour coding indicates the different hardness of the polishers.

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Polishers with contra-angle shank and spray guard, without abrasive media for the removal of dental plaque, with polishing paste. The integrated spray guard prevents polishing paste getting into the contra-angle. Optimal motor speed = 1,500 rpm. Colour coding indicates the different hardness of the polishers.

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Snap-on polishers with spray guard, without abrasive media for the removal of dental plaque, with polishing paste. Used in combination with the snap-on mandrel 4037RA. Optimal motor speed = 1,500 rpm. Colour coding indicates the different hardness of the polishers.

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Prophylactic treatment, maximum cleaning and polishing of the tooth surface

Healthy teeth and healthy gingiva – with the correct prophylaxis. The new prophylaxis brushes with improved polishing properties clean the tooth surfaces in preparation for subsequent polishing. This ensures thorough and reliable tooth cleaning. Colour coding indicates the different hardness of the polishers: white = standard brushes for removing dental plaque and polishing, pink = soft brushes for removing dental plaque in the region of the sulcus, to protect the soft tissue, yellow = hard brushes for removing heavy dental plaque and discoloration. For single use, non-autoclavable.

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Nylon brush with contra-angle shank for removing dental plaque, with polishing paste. The flexible nylon brushes also allow cleaning below the gingival margin. Colour coding indicates the different hardness of the brushes. Optimal motor speed = 1,500 rpm. For single use, non-autoclavable.

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Nylon brushes with screw thread for screw retention in all known prophylaxis contra-angle handpieces or in the reusable screw mandrel 4039RA. For single use with polishing paste.

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Nylon brush used in combination with the snap-on mandrel 4037RA. For single use with polishing paste.

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TopBrush nylon brushes with integrated polishing particles made from silicon carbide for the removal of heavy dental plaque without the use of polishing paste as well as for high-lustre polishing of composites, compomers and glass ionomer cement.

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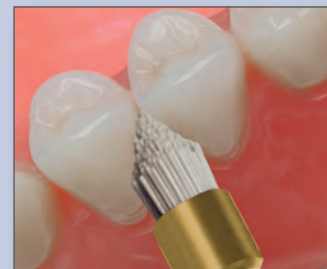
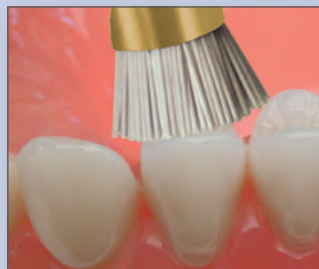
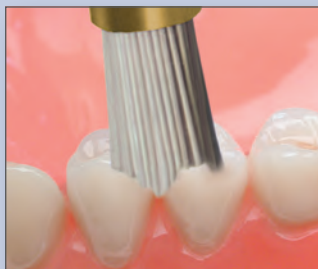
The special fibres contain silicon carbide (SiC) as the abrasive medium. This allows use without polishing paste. Each bristle is an individual polishing element.

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Functional principle: The polishing effect is created by the special brushes which incorporate an abrasive medium, silicon carbide (SiC). This allows use without polishing paste. The abrasive medium is used up during polishing and at the same time fresh SiC is constantly exposed on the surface of the fibre. This produces a uniform, consistently high polishing performance which also produces a quicker result.

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Amalgam restorations can only be removed with difficulty using diamond rotary instruments or standard tungsten carbide cutters and this produces a high heat build-up. The efficient cutting PowerCut drill with a special cutter tip ensures the removal of old amalgam fillings in the shortest possible time with minimal heat build-up.

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
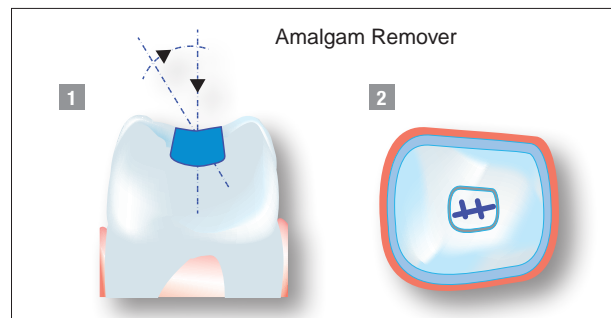
The cutting geometry with pyramid-shaped tip transversing blade (3) enables easy axial penetration in the amalgam filling, whereby the large chip escape channels ensure rapid cutting of the amalgam with minimal heat build-up.

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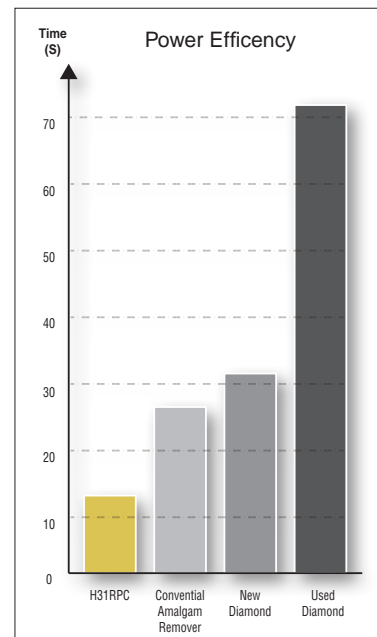Instructions for use: The PowerCut should be drilled into the filling axially or at an angle (1). Drilling should then be continued in a longitudinal and transverse direction (2) until the filling is removed. Recommended motor speed in a red contra-angle = 160,000 rpm or also in a turbine. When using as a crown remover (4), the porcelain veneer must first be removed using a diamond instrument.

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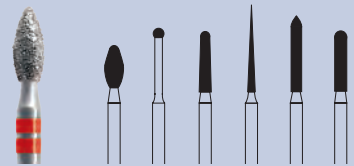
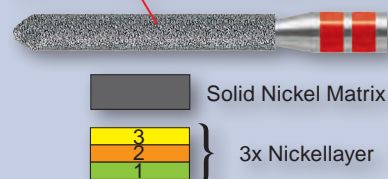
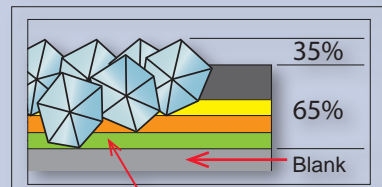
Preparing ZrO2 all-ceramic in the dental practice

To ensure a high service life of all-ceramic restorations it is important to prepare the sintered ceramic carefully to protect the material properties and to avoid microcracks and ceramic splitting off. Large areas should no longer be prepared, but only the necessary, minor fitting adjustments should be completed using special diamond instruments with water cooling. The grit size of the diamond instruments have been correspondingly adapted to avoid pulling out ceramic particles to ensure that there is no risk to the long-term stability of the all-ceramic.

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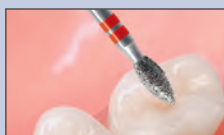
Occlusal Adjustment and Polishing

Fine-grit K-Diamond instruments are used for adjusting occlusal contact points. Ceramic polishers, CeraGloss are used for polishing.

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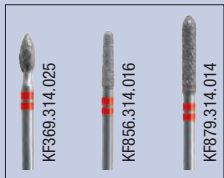
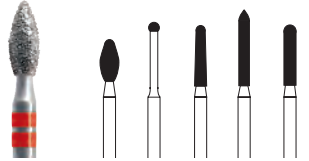
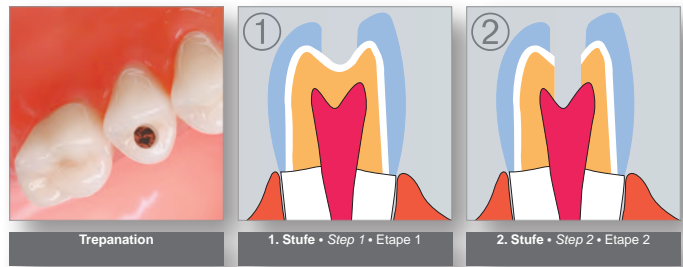
Trepanation in all-ceramic crowns

The preparation of an access cavity should be completed in 2 steps. (1) First the veneer porcelain should be removed using a K diamond. (2) The all-ceramic framework should then be perforated using a K diamond so that there is a distance of 0.5 mm to the veneer porcelain.

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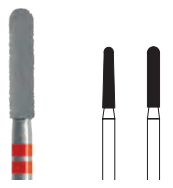
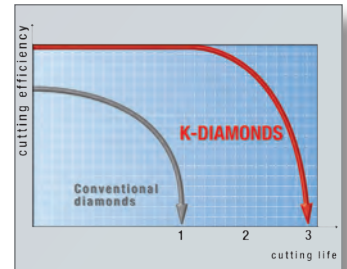
Crown remover for zirconia all-ceramic

The multilayer K diamonds guarantee a maximum service life on hard zirconia materials. A special bonding of the diamonds provides optimal cutting efficiency during separation with minimal vibration. Recommended motor speed = 160,000 rpm in a red contra-angle.

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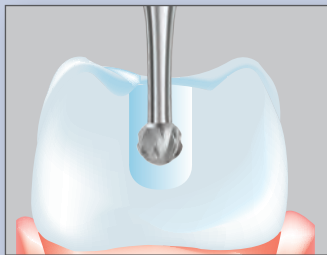
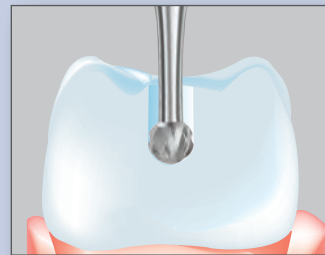
Excavation using the familiar C1S tungsten carbide bur also removes hard, healthy dentine.

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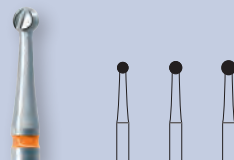
Minimally invasive excavation using the C1SDR tungsten carbide bur, removes only soft, carious dentine.

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Selective dentine caries remover

A further development of the C1S tungsten carbide bur, specially designed for excavating carious, soft dentine. The known advantages of the C1S, the fine, gentle material removal with high cutting efficiency can now also be transferred to the indication of excavation. The new C1SDR tungsten carbide bur is impressive due to its cutting efficiency in carious, soft dentine while it hardly removes any healthy, hard dentine. This is the perfect combination for minimally invasive excavation. A special blade design ensures vibration-free, gentle excavation.

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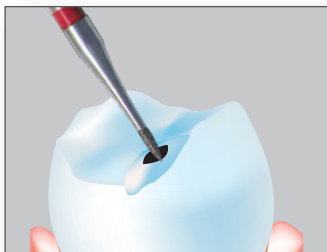
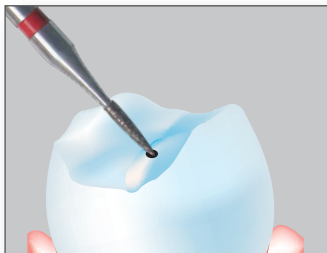
Minimally invasive preparation using microfine BAMBINO diamond instruments for selective removal of carious dentine while conserving the maximum amount of healthy tooth structure.

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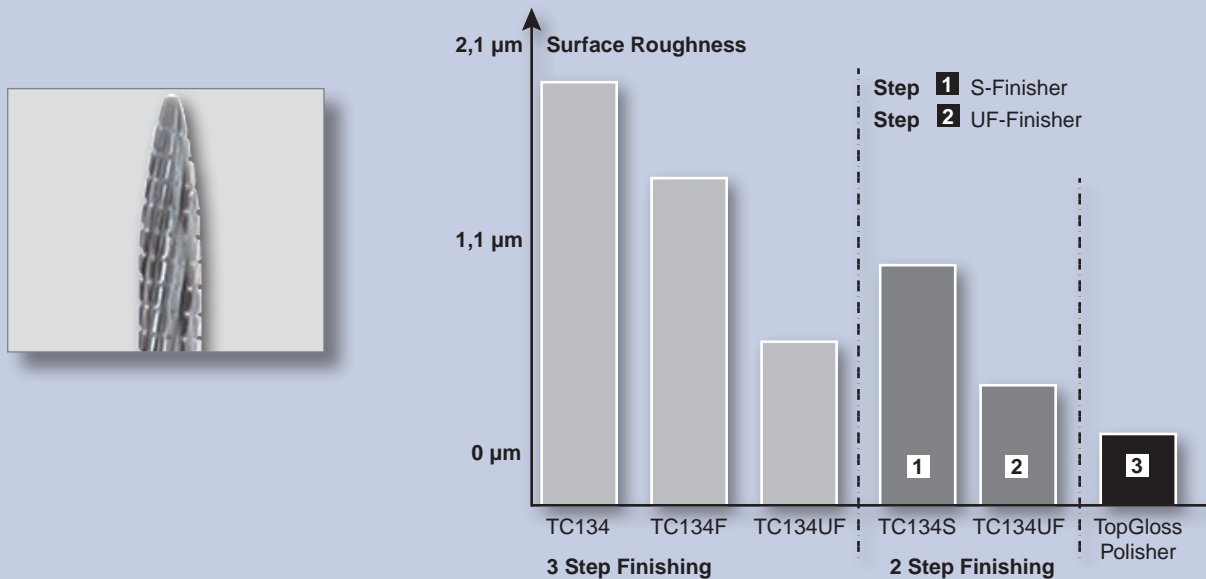
BAMBINO microfine diamond instruments for selective, minimally invasive preparation using specially designed instruments. The micro-small working diameter and the fine neck design enable selective preparation with minimally invasive contouring of the cavity, even with cavities that are difficult to access. The diamonds produce a smooth tooth surface, making further finishing of the preparation unnecessary.

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S-Finisher for economic preparation and optimal results on composite

Further development of the finishing blades with an additional spiral cross-cut has optimised the cutting performance and cutting quality of the tungsten carbide finishers for use on composite. The new cutting geometry in combination with the effective cutting angle, chip space depth and the angle of the blade ensures maximum cutting performance with a smooth cut, so that now only 2 finishing steps are required for the final preparation of a composite restoration. Step (1) = S Finisher, step (2) = Ultra-fine Finisher (white ring). High-lustre polishing is then completed using the TopGloss one-step diamond grit polisher (3).

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S Finisher, efficient and quick preparation

Finishing a composite filling is reduced from 3 to 2 steps using the new S Finishers.

Step (1) the S Finisher smooths the surface of the filling better than when using the 2 conventional red and yellow ring finishers.

Step (2) the ultra-fine white ring Finisher reduces the surface roughness to less than 1 µm and the surface is therefore ready for high-lustre polishing using the TopGloss diamond polisher.

Optimal motor speed = 20,000 rpm, with water spray cooling

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Efficiently cutting special endodontic bur used for detecting all root canals in the case of multiradical teeth. The H1SNL has a particularly high cutting efficiency and its long neck ensures a clear view in deeper regions of the cavity for opening the access cavity of the root canal system. Use in a contra-angle with micromotor at 1,500 – 2,000 rpm, without cooling, dry and apply only light pressure during preparation.

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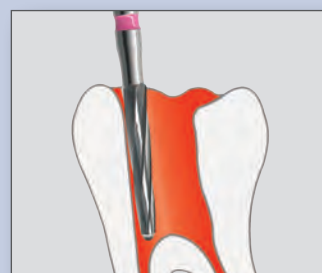
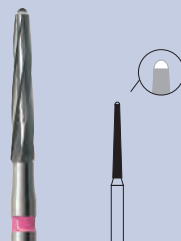
Special endodontic drill with a long, conical blade configuration for producing a funnel-shaped access into the pulp chamber. The rounded, safe-end drill tip prevents penetration into the root canal walls and the floor of the pulp chamber during drilling. Maximum motor speed 120,000 rpm, apply only light pressure during preparation.

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Crown Prep-Set, diamond instruments for the preparation of CAD/CAM fabricated ceramic crowns

CAD/CAM fabricated all-ceramic restorations require a ceramic-specific preparation with rounded edges and without sharp junctions. The appropriate preparation diamonds are contained in the Crown Prep set.

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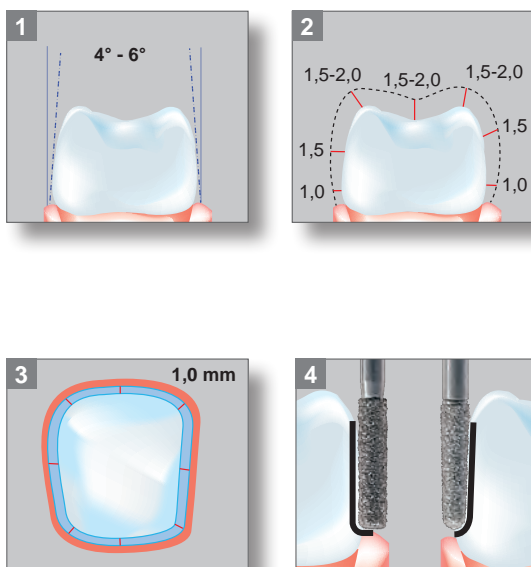
(1) Creating a tooth preparation with a taper angle of 4-6°. Round off all preparation junctions. (2) Reduce the external geometry of the tooth crown by 1.5 mm, the occlusal 1.5-2.0 mm and the margin 1.0 mm. (3) A shoulder preparation with rounded inner angle or a pronounced deep chamfer can be prepared. Preparation depth at the margin min. 1.0 mm. (4) Smooth the preparation margins using diamond finishers (red ring) that are congruent in shape.

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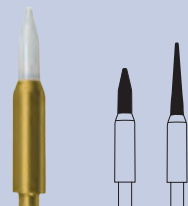
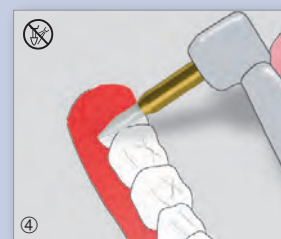
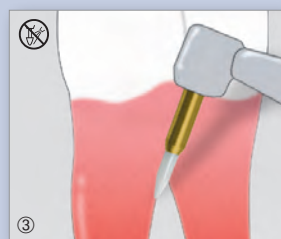
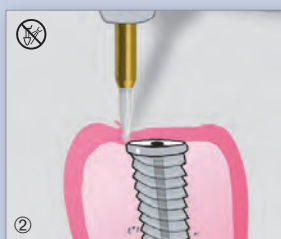
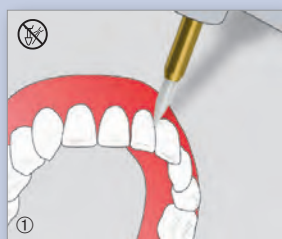
Soft Tissue Trimmer made from zirconia all-ceramic with a long, slender tip used as an alternative to a scalpel or electrosurgery. The high-performance ceramic tip allows all indications to be performed in gingivectomy using only one instrument. The long, slender tip ensures easy access during all applications in mucosa surgery.

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① Gingivectomy (eliminating hyperplastic gingiva) ② Recovering implants ③ Interradicular debridement to remove granulated tissue ④ Excising mucosa to expose impacted/partially impacted teeth

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Universal Bur Block

40600

Abmessungen / Dimensions 72 x 20 x 50mm
For 6 FG, HP or RA Instruments / max. length: 47mm

40610

Inhalt / Contents 8 Stk. / pcs.

40600 (without instruments)

Abmessungen / Dimensions 72 x 20 x 50mm
For 6 FG, HP or RA Instruments / max. length: 47mm

40601

Abmessungen / Dimensions 72 x 20 x 40mm
For 6 FG or RA Instruments / max. length: 37mm

40602

Abmessungen / Dimensions 61 x 25 x 30mm
For 12 FG or RA Instruments / max. length: 28mm

40603

Abmessungen / Dimensions 42 x 25 x 30mm
For 8 FG or RA Instruments / max. length: 28mm

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Bur blocks for the dental practice

The new bur blocks are manufactured entirely from stainless steel, making them suitable for all types of preparation. The instruments can be cleaned and disinfected in the block either in an instrument solution or also in an ultrasonic cleaner. They are then sterilised in an autoclave. The instrument shanks are retained in position by universal silicone plugs and so cannot fall out. FG or RA contra-angle instruments can be inserted in the silicone plugs.

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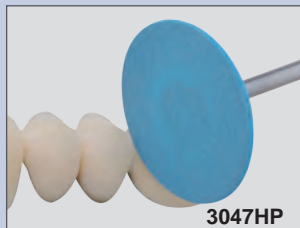
CeraGloss-FLEX high-lustre polishing system for all types of all-ceramic materials, ZrO2

New flexible diamond polishers for an optimum surface quality finish in 2 steps for finishing and high-lustre polishing on all types of ceramic/ all-ceramic materials. To obtain durable, high-lustre surfaces for natural, attractive ceramic restorations, specially for interdental regions and polishing contours. The flexible, thin polishers adapt perfectly to the anatomical surface contours. The CeraGloss FLEX polishers are made from highly filled silicone and special diamond grit as the abrasive polishing medium. For polishing results with minimal surface roughness; this reduces discoloration and plaque accumulation over the course of time. The high-lustre polished surfaces make a renewed glaze firing cycle unnecessary.

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Step 1 = Blue, polishing to smooth the surface
Step 2 = Yellow, high-lustre polishing without polishing paste

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Economic preparation of non-precious metal alloys with fine titanium blades VZ27

Generally, more material has to be removed during preparation of non-precious metals in the dental laboratory because of technical reasons during casting. The preparation instruments are therefore subjected to a large amount of stress and have only a short service life. New non-precious metal alloys are slightly softer, but are still difficult to machine. Cutters are consequently required that are as wear resistant as possible and at the same time also ensure high material removal rate. The newly developed fine titanium blade VZ27 has a very high cutting efficiency ensuring a high material removal rate while simultaneously smoothing the surface of the material, which then allows it to be easily polished.

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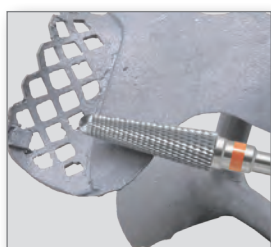
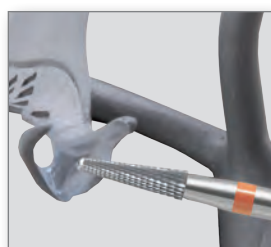
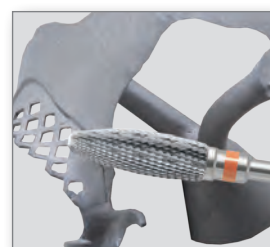
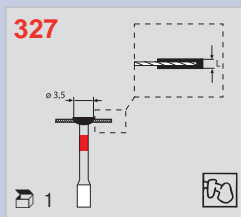
Diamond discs

New diamond discs fully coated on both sides for separating crowns and bridges from HiP ZrO₂ frameworks. The full diamond coating facilitates separation in areas that are particularly difficult to access. The special high-grade diamond coating ensures efficient material removal and increases the service life of the diamond discs. This guarantees rapid separation without splitting off on any type of all-ceramic materials. In addition, the particularly thin discs facilitate separation and contouring of ceramic restorations.

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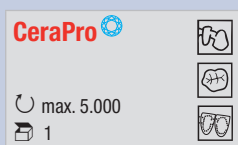
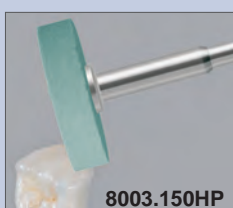
Trimming the sprue contact points and preparing the all-ceramic surface using the CeraPro wheel 8003.150HP. The ceramic trimmer with diamond grit is ideal for preparation without heating the framework.

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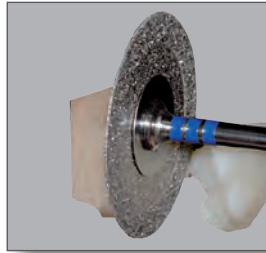
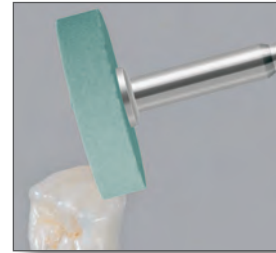
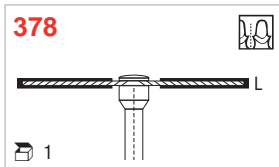
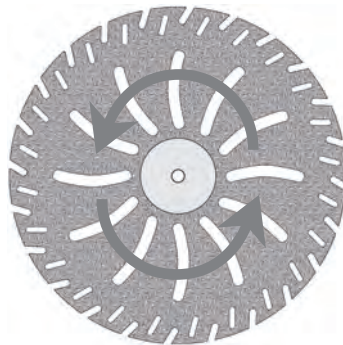
Time-saving and material-protecting devesting of pressable ceramic frameworks

New diamond disc fully coated on both sides for separating and cutting off of press moulds. Pressable ceramic investments must have a high stability. This property, however, makes it difficult to devest and the diamond disc may become jammed. The newly developed diamond disc was therefore provided with segmentation, which transports the investment towards the outside, preventing jamming or catching of the disc. The flange design provides high stability, guaranteeing a straight cut.

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