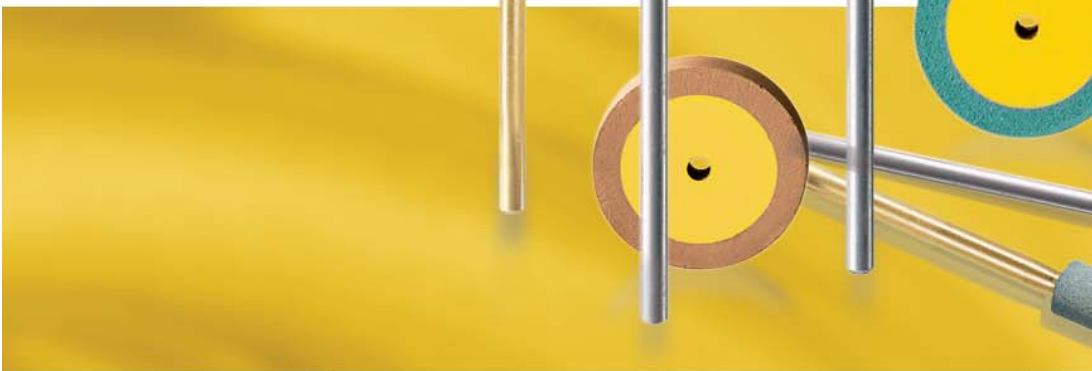
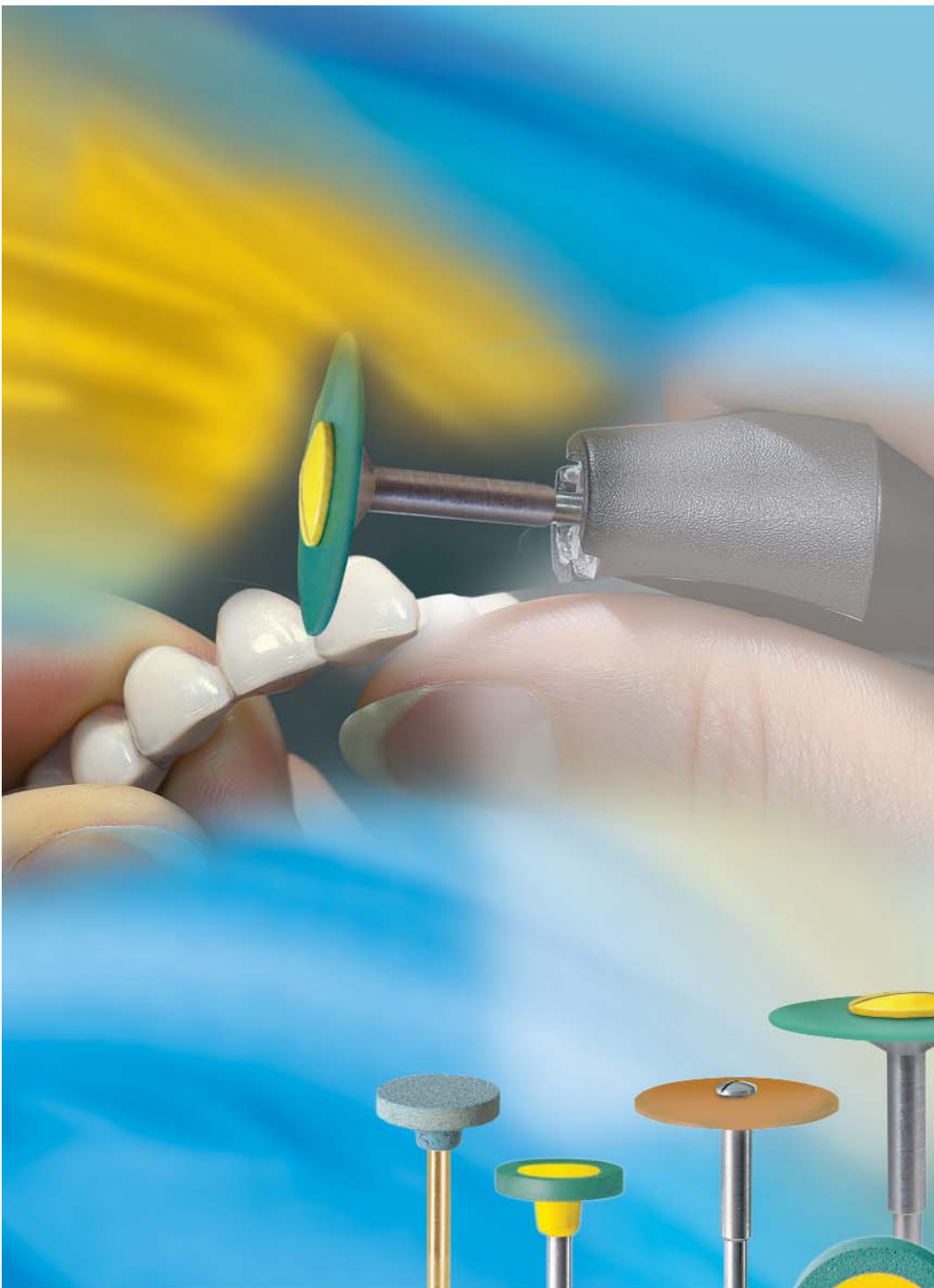
**Rotary Grinding and
Polishing Instruments**

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At your service!

You can contact our head office
 Mon - Thur 8 a.m. to 5 p.m.
 Fri 8 a.m. to 3 p.m.
 by e-mail, telephone or telefax:
 info@eve-rotary.com
 Telephone +49 (0) 7231 9777-0
 Facsimile +49 (0) 7231 9777-99

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Instruction for use / Safety Tips / Sterilization

Instructions for use, safety and
 sterilization can be found on our website:
www.eve-rotary.com/downloads

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Controlled quality and tested safety

EVE Ernst Vetter GmbH is DIN EN ISO 9001
 certified. The high quality standards in deve-
 lopment, production and administration as
 well as in distribution are laid down by the
 Quality Management System. The medical-
 products are also certified in accordance with
 DIN EN ISO 13485. They meet the require-
 ments of Council Directive 93/42/EEC issued
 on 14 June 1993 concerning medical pro-
 ducts, as well as the regulations contained in
 the Medical Devices Act (MDA) issued on
 2 August 1994, and are CE-labelled. Strict
 incoming goods inspections for raw materials
 and state-of-the-art methods of production
 guarantee the consistently high quality of all
 EVE products.

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Quality, precision and innovation will
 continue to be the distinguishing features
 of our products and our company.

	Seite / Page
Zirkonoxid in Spiegel-Qualität / Mirror polishing for zirconia	3 - 4
Implantatprothetik / Implant prosthetics	5 - 6
Schleifkörper / Abrasive Grinders	
EVE DIASynt PLUS	7
Polierer / Polishers	
EVE DIACERA	8

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Key skills in surfacing

Surfacing plays a decisive part in dentistry. Whether smooth or structured surfaces, surfacing today's increasingly demanding materials requires increasingly efficient tools. At the moment, users focalize on oxide ceramics, a product category that has become in the last years a world standard for aesthetical and highly resistant restorations.

EVE Ernst Vetter GmbH has been a specialist in dental surfacing for over 40 years now, mostly in the field of conservative and prosthetic dentistry – for all steps from grinding to high polishing. The new EVE generation of abrasive grinders and polishers enables you to surface hard zirconia (ZrO_2) and thus use it for making definitive prosthetic restorations, with no need for a veneering ceramic.

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This brochure shows you application examples for surfacing of oxide ceramics.

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EVE abrasive grinders and polishers for zirconia dental restorations deliver high polishing quality in an economical way

Zirconia ceramic – a material as resistant as metal – can be used for making crowns, bridges, inlay bridges, implant crowns and suprastructures. These restorations are fabricated in a subtractive way, using CAD-CAM systems. Zirconia is similar to tooth color and can be dentin colored. It is now possible to use this material for making full-sized and restorations with cusps, fissures and contact points, which means that ceramic veneering – a time consuming step – is no longer needed. Zirconia restorations featuring biogeneric and functional occlusal surfaces are a technically perfect and cost effective way of making durable restorations at an affordable price for the patient.

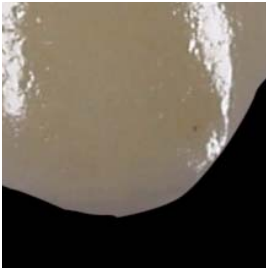
Full ceramic restorations made of hard sintered zirconia can now be grinded and polished – without modifying neither the crystalline structure nor the martensitic phase of this material – by using high performance EVE DIACERA diamond polishers and EVE DIASYNT Plus synthetically bonded grinders. The result obtained: mirror polished restorations which push back the bacterial plaque and resist to exogene discolorations caused e.g. by tee, coffee, nicotine or lipstick.

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Embrasures of pontics need no veneering ceramic, just polishing. The bridge connections can therefore be of maximal size for optimal resistance to mastication forces.

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
ZrO₂ crown after stain application. EVE polishing of the thin and precise crown contour for better resistance to bacterial plaque.

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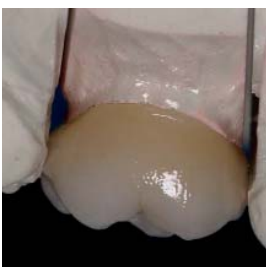Maryland (adhesive) bridge. EVE polished ZrO₂ retainers. In order to avoid chipping, glaze application is not recommended.

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Full-sized ZrO₂ crown after grinding, Al₂O₃ sandblasting, cervical stain and glaze firing. Without framework and veneering ceramic, and thus economical.

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Pontic with anti-rotation device made of ZrO₂ shown after EVE polishing. Interproximal contacts are checked using dental floss.

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9 unit bridge, dentin colored. Due to the full-sized construction, connections can be bigger for better fracture resistance, and no veneering ceramic is needed, which means no chipping. All outer surfaces and pontics have been EVE polished.

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EVE Diasynt Plus und Diacera for implant prosthetics

University studies showed that zirconia could be successfully used also for implant abutments and mesostructures as well as for implant supported crowns or bridges. The smooth polished zirconia surfaces enhance, together with the high biocompatibility of this material, gingival regeneration. As regards screwed, anti-rotation abutments, the narrow joint between suprastructure and crown needs good polished surfaces.

For this application field too, namely the machining and polishing of zirconia surfaces, high performance EVE DIACERA diamond filled polishers and EVE DIASYNT Plus synthetically bonded grinders proved to be successful.

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Single implant crown. The collar and a part of the palatine surface have been EVE polished.

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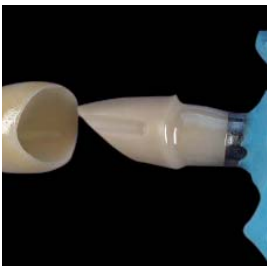
ZrO₂ implant bridge screwed on gingival mask. The whole lingual area has been EVE polished.

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Embrasures of a ZrO₂ dentin colored implant bridge without veneering ceramic and featuring proper EVE polishing.

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Anti-rotation ZrO₂crown and abutment. The subgingival abutment area has been EVE polished.

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Full-sized ZrO₂ implant crown without framework . The crown lower part has intensively been EVE polished in order to offer the marginal gingival a smooth contact surface and thus enhance soft tissue regeneration.

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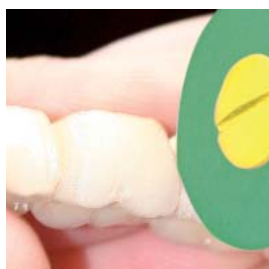
EVE grinders and polishers for zirconia restorations

Zirconia, also called “white steel”, was until now considered a material difficult to machine. But today’s prosthetic technology makes it possible to machine it into anatomical restorations, so that in many cases neither framework nor veneering ceramic is needed. Such ZrO₂ restorations being also more cost effective, they are the right answer to today’s requirements.

EVE proposes for this application field its high performance DIACERA diamond polishers and EVE DIASYNT Plus synthetically grinders, which eliminate milling marks without modifying the crystalline structure or martensitic phase of zirconia. The surfaces obtained with these tools are smooth, mirror polished and more resistant to bacterial plaque. EVE grinders are best suited to surface zirconia frameworks too. High performance

EVE grinders and polishers for zirconia:

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EVE DIASYNT PLUS

High performance, synthetically bound diamond abrasive grinders for ceramic, especially oxide ceramic

The high-temperature resistant bond with selected diamond grit makes it possible for Diasynt Plus to be used for a wide range of applications that demand high edge stability. It is particularly developed for the processing of highly resistant materials such as zirconia and aluminium oxide when its advantages compared to other abrasive grinders become apparent. Diasynt Plus guarantees extremely high removal capability with minimized heat development, thus greatly protecting the material during use and lessening the danger of microcrack formation.

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EVE DIACERA

Two-step diamond polishing system for ceramic, especially tailored for CAD/CAM restorations and oxide ceramics.

Green = 1st step: Smoothing and pre-polishing
Orange = 2nd step: High-lustre polishing

These polishers filled with selected diamond grit made especially to work together are ideally suitable for adapting and correcting CAD/CAM restorations. The processing and polishing of framework ceramics is easily possible.

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EVE America, Inc.