Plastiform

**Plastic patterns for modelling partial framework.
Precision and perfect reproduction of details.**

Thanks to its composition, its manufacturing process, as well as its quality, the Plastiform are the ideal plastic patterns for the construction of partial denture framework.

The excellent resistance to deformation and its good adherence to the models are highly appreciated by the dental technicians.

The Plastiform can be used on a cold model, by using the Colform adhesive liquid, or on a hot model (just warm up the model to 38/40 °C) without adhesive.

Keep the Plastiform patterns in the box at a temperature of about 20 °C (shelf life - 2 years).

No residue of plastic will be found inside the investment, upon complete heating in the burnout furnace.

To obtain the best results, we recommend:

- To avoid contact with greasy substances on the surface of the model.
- To heat up the model to about 38/40 °C. At this temperature the patterns will stick to the model.
- The use of the liquid hardener Duraform is strongly recommended.

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