

**Ready for use, fireproof Material
for the Manufacture of individual Firing Trays.**

Easy-Fix™ is a ready for use, fireproof material for the production of individual firing trays.

With its soft, creamy consistency it has exceptional processing characteristics, which enable the technician to fix inlays, onlays, veneers, crowns and bridges on pins or directly on the firing tray.

Pressed ceramic objects on which cracks occurred, can be safely fixed on the firing tray and put into the oven for a correction firing. The fitting accuracy is retained, warping of shoulders and margins during firing are a thing of the past.



ZUBLER Easy-Fix™

3 Syringes

Item-No. 20/0550



ZUBLER Easy-Grip™

Item-No. 20/0500



Repair of pressed objects




Safe holding with **Easy-Grip™**



Fill with **Easy-Fix™** Paste



Completely fixed firing object

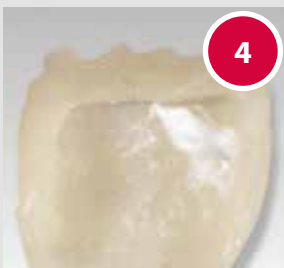

Repaired pressed object after firing

Fixing of all-ceramic objects



Put in **Easy-Fix™** Paste



Put on firing tray carefully



Pressed object after firing



Easy removal of **Easy-Fix™**

Fixing of metal ceramic objects




Ceramic shoulder before firing



Fill with **Easy-Fix™** paste



3-unit-bridge prepared for firing

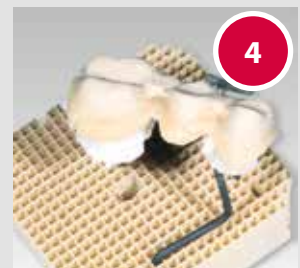

Bridge after firing

Distributed by:

CE



B00147-01/2018